


Dental

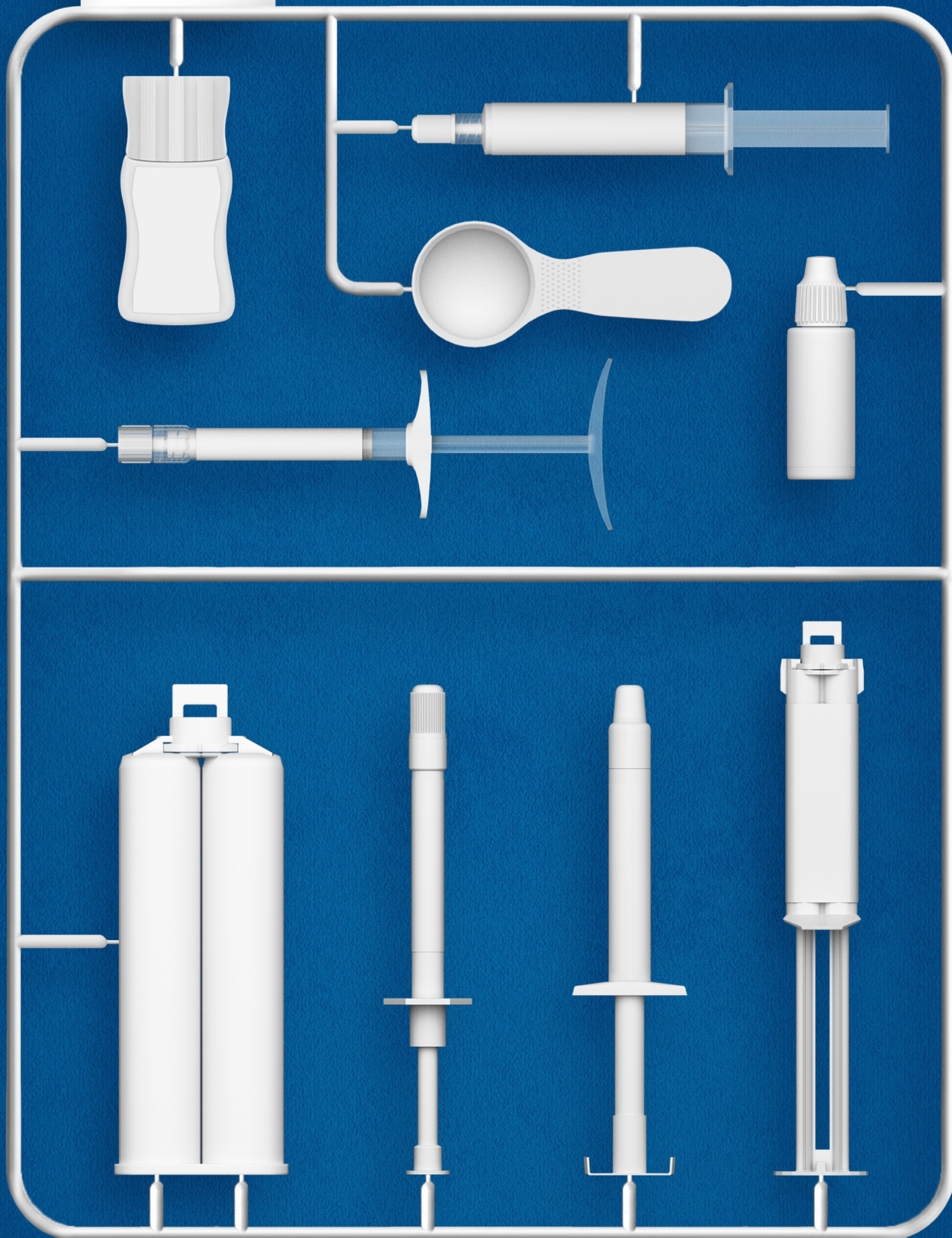

Bright Resin
Restorative Material



Dentium

Contents

Bond & Restoratives	04
Universal Bond	
Flowable Resin	
Bulk Fill Flowable	
Etchant	
Core	
Cements	09
Resin Cement 2	
Zirconia Primer	
Orthodontic Adhesive	
Impression materials	12
Impress (Light, Medium, Heavy, Bite, Putty)	
MTA materials	13
MTA Sealer Plus	
MTA LINER	
3D Printing materials	14
Model	
Surgical Guide	
Flexible Denture	
Clinical cases	19

Bond & Restoratives

Total etching

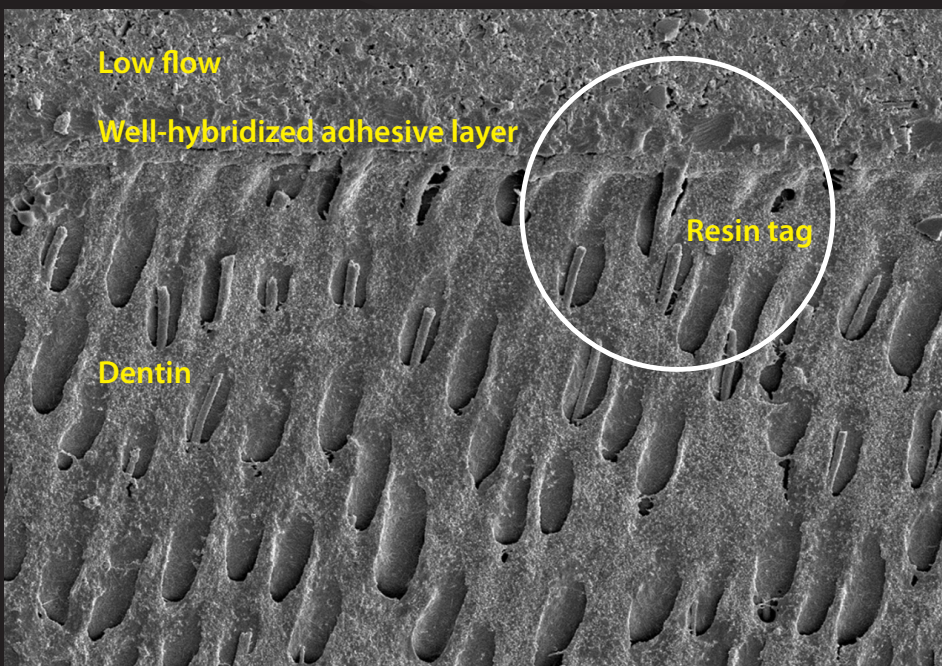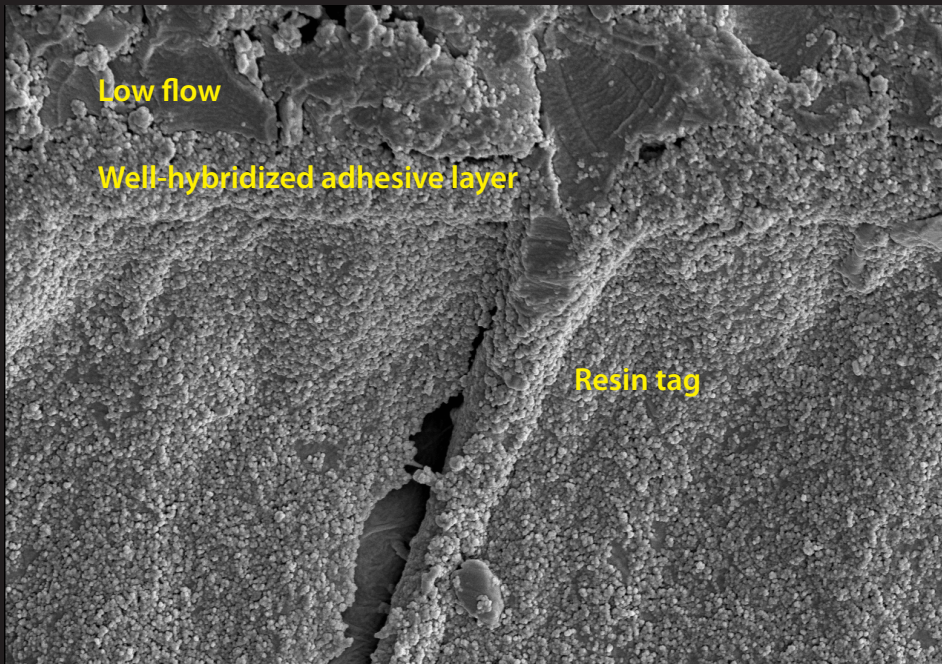

Procedure

Etchant (Selective & Total etching)¹ ▶ Wash and Dry ▶ Universal Bond² ▶ Dry
 ▶ Light-cure ▶ High/Low/Bulk flow³ ▶ Light-cure ▶ Polishing

Case 1. Diastema



Pre-treatment



1



3



Post-treatment

Case 2. Anterior fracture



Pre-treatment



1



3



Post-treatment


Case 3. Cervical abrasion



Pre-treatment



1

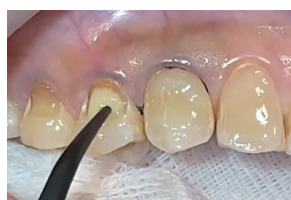

3



Post-treatment

Tip

- Recommended etching time for dentin: ≤ 15 seconds.
(Excessive etching of dentin may cause postoperative sensitivity and decrease in bond strength)
- Wash and dry at once by etching dentin and enamel with time interval.
- For efficient penetration of the adhesive to tooth surfaces, agitate for 20 seconds with a gentle pressure using a micro-brush.
- Dry the agitated surface thoroughly with air till the adhesive does not move.

Universal Bond


Single bottle, light-cured dental adhesive

Strong (10-MDP, 4META) Strong and durable bonding


Film thickness Low film thickness, $\leq 3\mu\text{m}$



Strong and durable bonding ¹

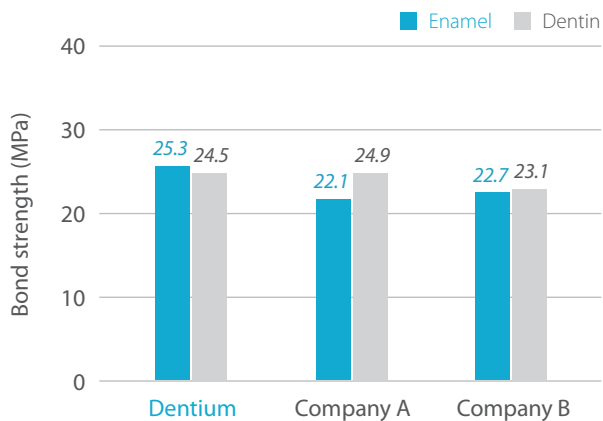

Low film thickness


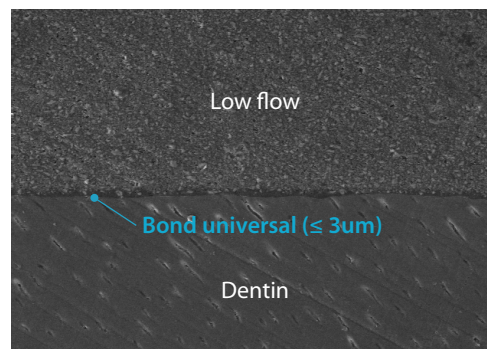


Flowable Resin

Single bottle, light-cured dental adhesive

Convenience Two kinds of viscosity (High/Low)

High radiopacity

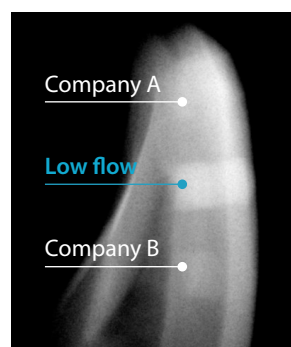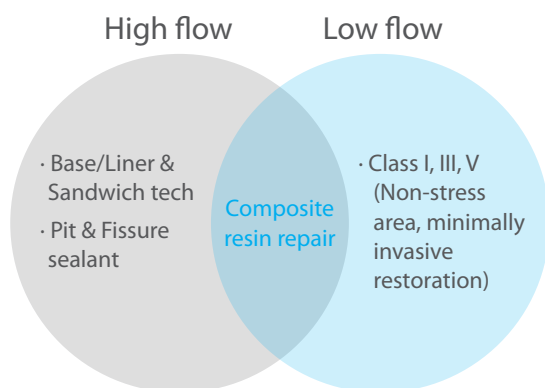

Bulk Fill Flowable

Light-cured flowable composite




Self-leveling Good cavity adaptation

Convenience High depth of cure & Low shrinkage


Product features

Properties	Value	Method
Shade	Universal, A2	-
Polymerization shrinkage(%)	2.14	-
Depth of cure(mm)	7.0	ISO 4049
Flexural strength(MPa)	125	ISO 4049
Fluoride release	Release	-


Cavity adaptation

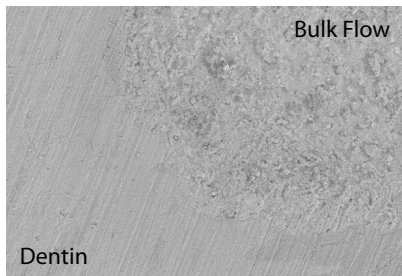

High depth of cure¹

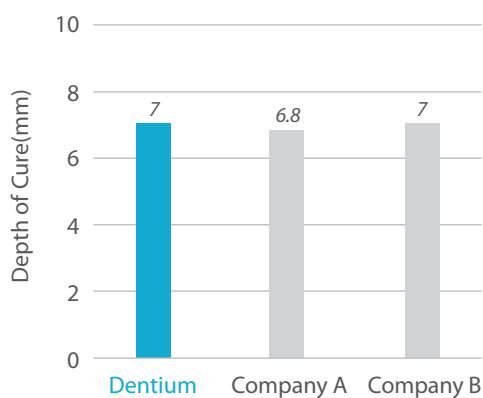

Low polymerization shrinkage²

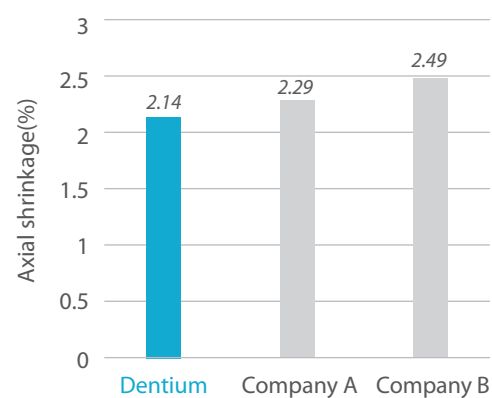

Etchant

Dentin/Enamel etching gel


Convenience Selective or total etching

Enhancement Improves bonding strength




Clear etched surface

Enamel (15~30sec)

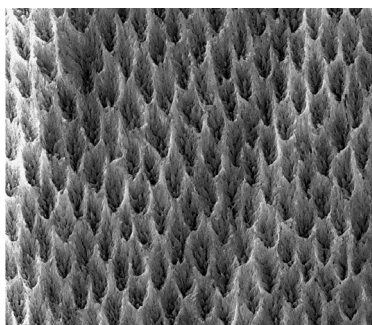

Dentin (10~15sec)

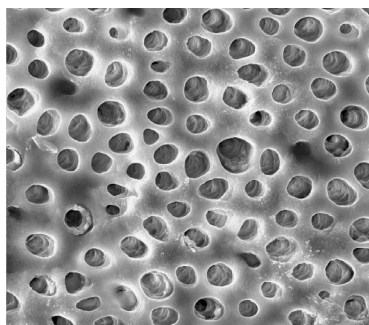

Core


Dual-cured core build-up composite

Convenience Excellent light-cure depth and high radiopacity

Preparation Dentin-like cuttability to ensure controlled




High depth of cure ¹

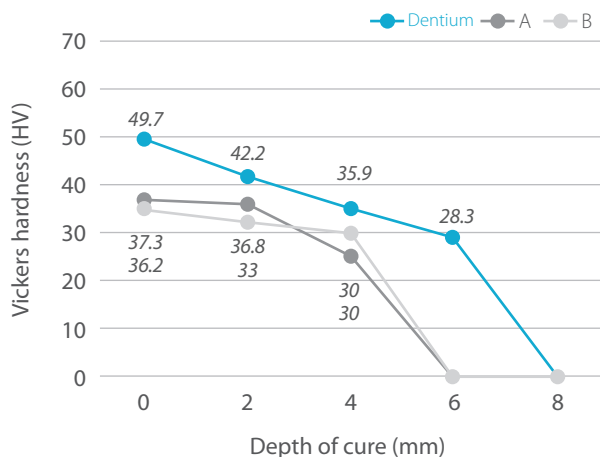


1. GENOSS Data on file

Excellent radiopacity

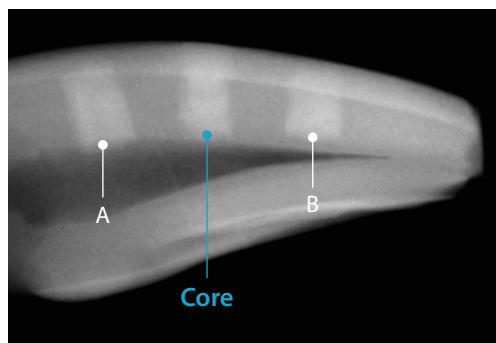

Cements

Procedure

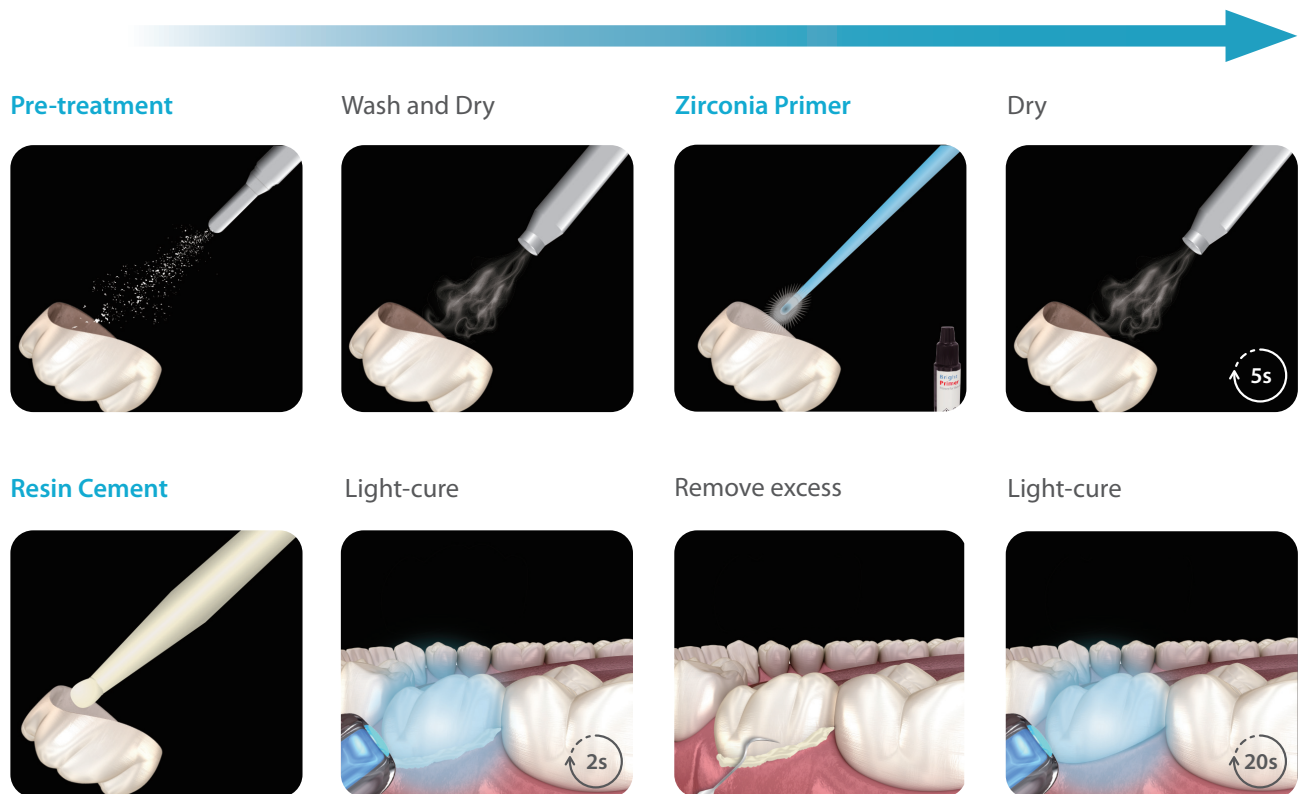

Tip

- For higher bond strength, the restoration should be pre-treated prior to cementation.
 - Zirconia & Metal: sandblast with 50 μm Al_2O_3 for 15 sec at 2.5 bar
 - Glass ceramic: etch with 5% hydrofluoric acid for 20 sec
- Application of Bright Primer containing 10-MDP further improves the bond strength.
- After applying Bright Resin Cement 2, place the restoration within 2 min and light cure for 2 sec to remove excess.

Resin Cement 2

Self-adhesive resin cement




Stable Stability Stable in both refrigerated and room temperature storage

Strong (10-MDP) Strong bonding to indirect restoratives

Product features

Properties	Value	Method
Shade	Universal, Opaque	-
Flexural strength(MPa)	121	ISO 4049
Bond Strength (MPa)	23.2 (@ Zirconia)	ISO 29022
Film thickness (μm)	10.7	ISO 9917
Working/Setting time (min)	≥ 2:30/≤ 4:00 (23±1°C)	ISO 4049

Strong bonding to indirect restoratives ¹

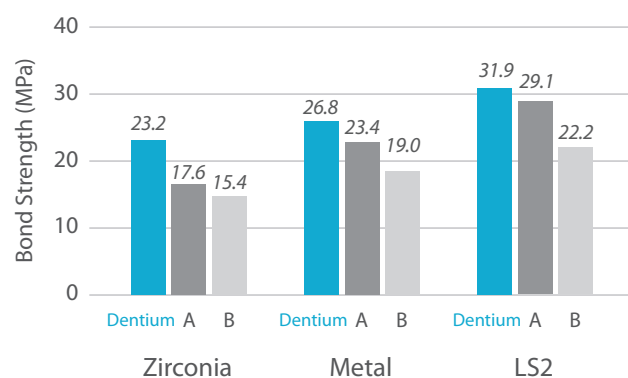

Primer


Primer for indirect restoration

Universal Universal primer containing 10-MDP

Improve Improves bond strength between cement and restorations



Stronger bonding to indirect restoratives ²

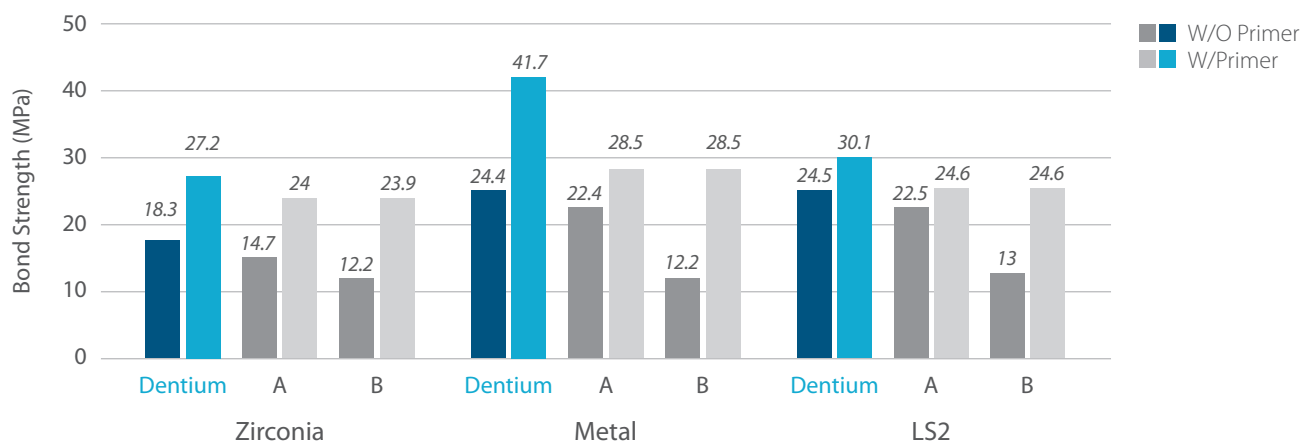

1,2. GENOSS Data on file

Orthodontic Cement

Light cure orthodontic adhesive

Strong (10-MDP)



Stable and high bond strength between enamel and brackets

Simple Simple orthodontic bonding procedure (No priming step)





Simple yet strong bonding ¹

Uniform bonded interface

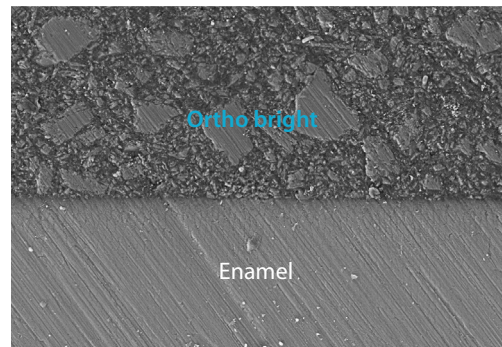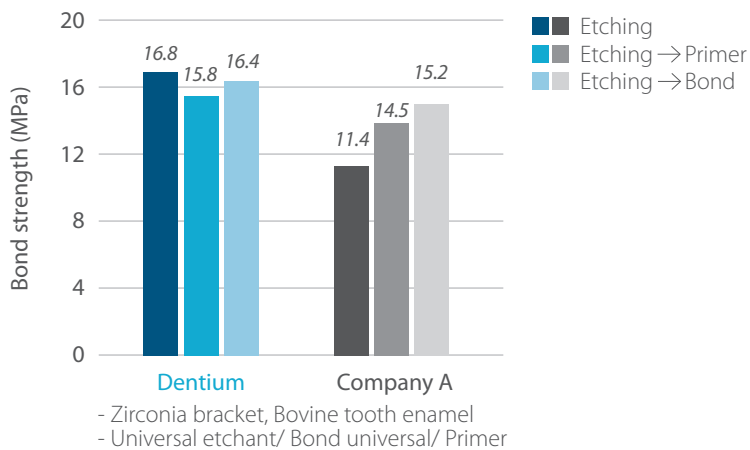

Strong bonding to enamel and zirconia brackets ²

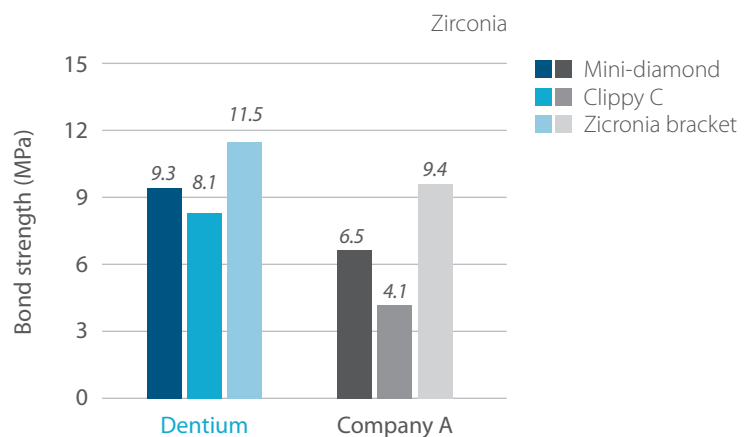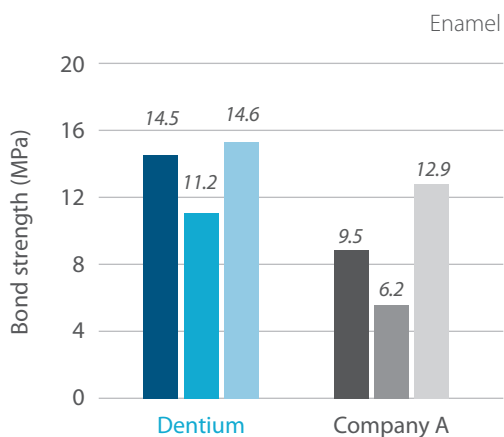

Impression materials

Impress


Vinyl polysiloxane impression material

Accuracy Highly accurate and void-free impression


Convenience Superior hydrophilicity and mechanical properties




Mechanical properties ¹

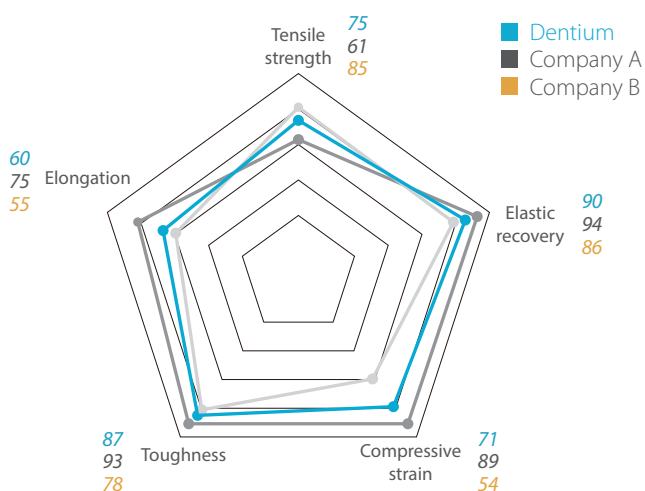



Light/ Medium/ Heavy/ Putty

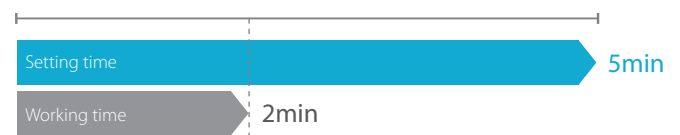

Bite

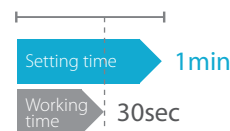

Accurate and detailed impression

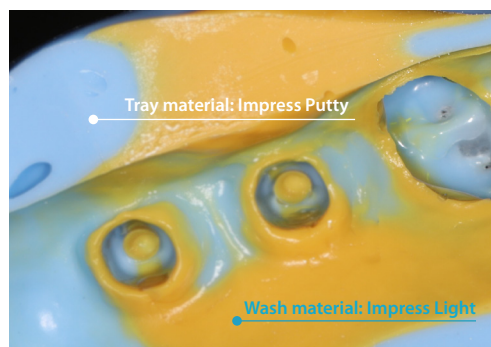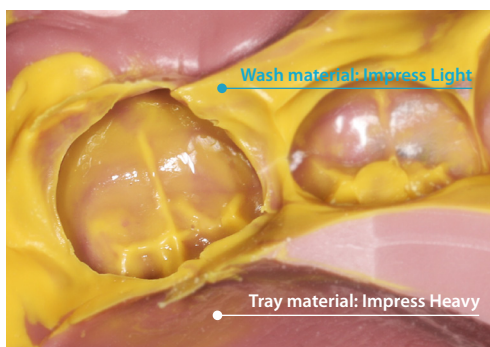

1. GENOSS Data on file

MTA materials

MTA Sealer Plus


Root canal sealing material

Anti-bacterial effect Impervious sealing with antibacterial effect


Convenience Paste-type calcium silicate based endodontic sealer




Setting time ¹

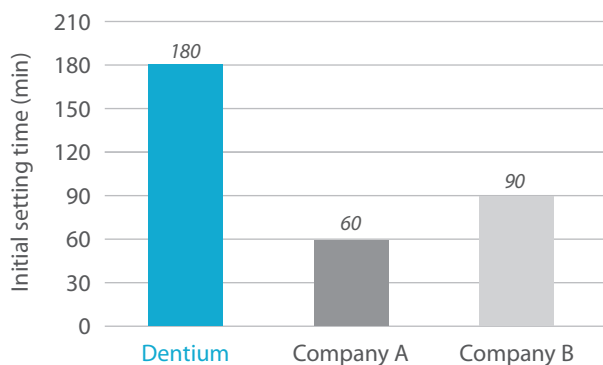

Flowability ²

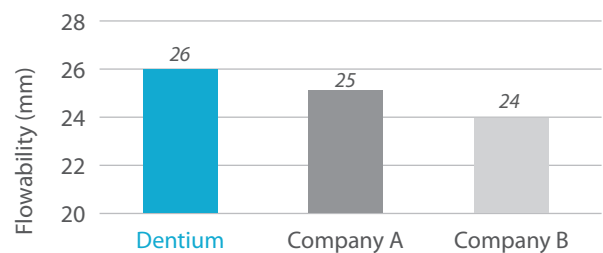

Radiopacity

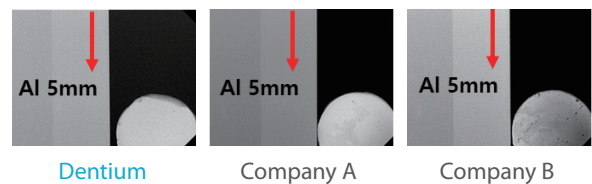

MTA LINER

Pulp Protectant/Liner



Pulp protection

Light curable, resin-modified calcium silicate for pulp protection


Convenience Controlled calcium release and high radiopacity



High radiopacity ³

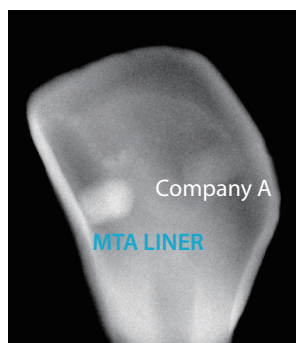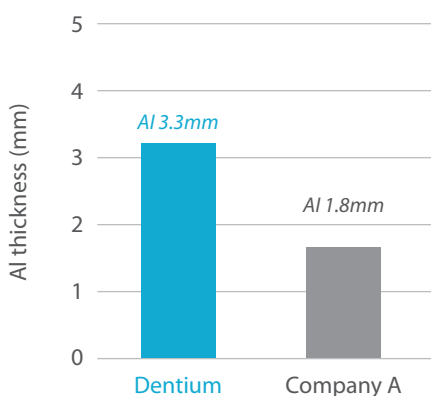

Excellent sealing

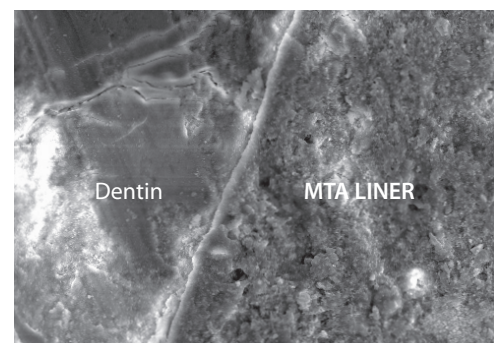

3D Printing materials

Model

3D dental model



Beige color Mechanical Properties

High mechanical properties enable various uses

Specification

Properties	Value Method	Method
Color	Beige	-
Viscosity	≤ 500 cP	BrookField
Flexural strength	≥ 120 MPa	ASTM D790
Hardness	≥ 80 Hs	ISO 868



Surgical Guide

Guide template for implant surgery



Transparency

High transparency improves surgical accuracy

Stability

Excellent discoloration resistance and fracture resistance

Specification

Properties	Value Method	Method
Color	Clear	-
Viscosity	≤ 400 cP	BrookField
Flexural strength	≥ 130 MPa	ASTM D790
Flexural modulus	$\geq 2,000$ MPa	ASTM D790



Flexible Denture

3D Flexible Denture



Light cured denture resin (A2/ A3/ Pink Shade)

High fracture resistance
and durability enable various cases



Specification

Properties	Value Method	Method
Color	A2/ A3/ Pink	-
Viscosity	≤ 700 cP	BrookField
Flexural strength	≥ 65 MPa	ISO 20795-1
Hardness	≥ 80 Hs	ISO 868

Bond & Restoratives

Product	Package	Model no.
Universal Bond	6ml x 1ea	BU6
Low Flow	2ml x 1ea	LFA1/ LFA2/ LFA3/ LFA3.5/ LFA4 LFOA2/ LFOA3/ LFCV/ LFBW
High Flow	2ml x 1ea	HFA1/ HFA2/ HFA3/ HFA3.5/ HFA4 HFOA2/ HFOA3/ HFCV/ HFBW
Bulk Fill Flowable	2g x 1ea	BFUS/ BFA2S
Etchant	5ml x 2ea	E5
Core	7.7g x 2ea	BCN

Cements

Product	Package	Model no.
Resin Cement 2	7.6g x 1ea	BRCU/ BRCO
Zirconia Primer	2ml x 1ea	P2
Orthodontic Adhesive	2ml x 1ea	OB4G

Impression materials

Product	Package	Model no.
Impress-Light	50ml x 4ea	DIM50LBF-40
Impress-Bite	50ml x 4ea	DIM50BT-40
Impress-Medium	50ml x 4ea	DIM50RBF-40
Impress-Heavy	50ml x 4ea	DIM50HB-40
Impress-Putty	500g x 2ea	DIM50PTF-40

MTA materials

Product	Package	Model no.
MTA Sealer Plus	1g x 1ea	MTA1
MTA LINER	2g x 1ea	C-4001P

3D Printing materials

Product	Package	Model no.
Model	1kg x 1ea	B3DMD
Surgical Guide	1kg x 1ea	B3DSG
Flexible Denture	1kg x 1ea	FD-PI/ FD-A2/ FD-A3

Handling Precautions

Light-cure products

Universal Bond, High/Low Flow, Primer, Orthodontic Cement, MTA LINER

- Keep in cool and dark places to avoid direct sunlight and moist conditions.
- In case of Bond and Primer, keep the cap close tightly to prevent volatilization after use.

Dual-cure products

Resin Cement 2, Core

- Store at 2 ~ 8 °C when not in use and keep at ambient temperature right before use.
- When using a syringe for the first time, squeeze out a small amount of material.
- Temperature affects working and setting times.

Impression materials

- When using a cartridge for the first time, squeeze out a small amount of material and then install a mixing tip.
- Temperature affects working and setting times.

MTA Sealer

- Keep in a pouch without removing a disposable tip after use.
- For reuse, squeeze out a small amount of material and replace with a new tip.

Clinical Cases

www.clinicaldentium.com



Bright Resin

Restorative Materials

Specifications are subject to change without any notice.
Some products listed in this catalog are not available in the market due to pending approval.

Dentium *GENOSS*

HEAD OFFICE

10F, 21, Teheran-ro 87-gil, Gangnam-gu, Seoul, Korea Tel +82-80-080-2875 Fax +82-2-501-9560

CAT-G-BR(Rev.4,2407)

MANUFACTURER

1F, Gyeonggi R&DB Center, 105 Gwanggyo-ro, Yeongtong-gu, Suwon-si, Gyeonggi-do, Korea (16229)
12F, Gwanggyo Business Center, 156, Gwanggyo-ro, Yeongtong-gu, Suwon-si, Gyeonggi-do, Korea (16506) T +82-31-888-5100

HOMEPAGE

www.dentium.co.kr / www.genoss.com